

Transducer Disinfectant / Pre-Cleaner / Gel Compatibility Table

Rev. 10

○: FDA Approved X: Not Approved(DO NOT USE) Δ: Not Tested (DO NOT USE)
S = Spray; W = Wipes; L = Liquid; P = Powder; G = Gel

Product Type	Disinfectant								
	Cidex OPA	Gigasept FF	Virkon	Cidex Plus	Wavicide-01	AIDAL PLUS	Cetylcode-G	Sporicidin	Super Sani-Cloth
	L	L	L	L	L	L	L	L	W
C1-6	O	O	O	O	O	O	O	O	O
C1-6T	O	X	O	O	Δ	O	Δ	X	O
C1-6i	O	O	O	O	O	O	O	O	O
SC1-6	O	O	O	O	O	O	O	O	O
C5-8	O	O	Δ	O	O	Δ	Δ	Δ	Δ
SC1-4HS	O	O	O	O	O	O	O	O	O
SC1-4H	O	O	O	O	O	O	O	O	O
SC1-6H	O	O	O	O	O	O	O	O	O
L3-8	O	O	O	O	O	O	O	O	O
L3-8i	O	O	O	O	O	O	O	O	O
L3-12	O	O	O	O	O	O	O	O	O
L3-12T	O	X	O	O	Δ	O	Δ	X	O
L3-12i	O	O	O	O	O	O	O	O	O
L3-12H	O	O	O	O	O	O	O	O	O
L3-12HWD	O	O	O	O	O	O	O	O	O
L3-12X	O	O	O	O	O	O	O	O	O
L8-17	O	O	O	O	O	O	O	O	O
L8-17i	O	O	O	O	O	O	O	O	O
L8-17X	O	O	O	O	O	O	O	O	O
MP1-5X	O	O	O	O	O	O	O	O	O
SP1-5	O	O	O	O	O	O	O	O	O
SP1-5i	O	O	O	O	O	O	O	O	O
SP3-8	O	O	O	O	O	O	O	O	O
SP1-5X	O	O	O	O	O	O	O	O	O
VC1-6	O	O	O	O	Δ	Δ	Δ	Δ	O
SVC1-6	O	O	O	O	Δ	Δ	Δ	Δ	O
SVC1-6H	O	O	O	O	Δ	Δ	Δ	Δ	O
E3-10	O	Δ	O	O	Δ	Δ	Δ	Δ	O
EN3-10	O	Δ	Δ	O	O	Δ	Δ	Δ	O
EV3-10	O	O	O	O	O	O	O	O	O
EV3-10T	O	O	O	O	O	O	O	O	O
EC3-10	O	O	O	O	O	O	O	O	O
EC3-10T	O	O	O	O	O	O	O	O	O
E3-10H	O	Δ	O	O	Δ	Δ	Δ	Δ	O
EC3-10H	O	O	O	O	O	O	O	O	O
EV3-10H	O	O	O	O	O	O	O	O	O
IO3-12	O	O	O	O	O	O	O	O	O
CW2.0	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
CW5.0	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
SVC1-6	O	O	O	O	Δ	Δ	Δ	Δ	O
IO8-17	O	X	O	O	Δ	O	Δ	X	O
EV3-10X	O	X	O	O	Δ	O	Δ	X	O
EC3-10X	O	X	O	O	Δ	O	Δ	X	O
VE3-10H	Δ	Δ	Δ	O	O	O	Δ	Δ	Δ
C5-8N	O	X	O	O	Δ	O	Δ	X	O
C5-8NT	O	X	O	O	Δ	O	Δ	X	O
SP1-5T	O	O	O	O	O	O	O	O	O
SP3-8T	O	X	O	O	Δ	O	Δ	X	O
IO8-17T	O	X	O	O	Δ	O	Δ	X	O

※ CAUTION ※

! Do not wipe the cable, strain relief, and connector of the transducer with the disinfectant, as it can damage and/or discolor those parts of the transducer.
! Do not immerse transducers for longer than one hour, unless they are sterilizable. ☒

Global ALPINION Headquarters

Verdi Tower, 72, Digital-ro 26-gil, Guro-gu, Seoul, 152-848, Korea [Homepage / www.alpinion.com](http://www.alpinion.com)

e-mail / marketing@alpinion.com

TEL +82-2-3282-0900 FAX +82-2-851-5591

Copyright©2016 ALPINION MEDICAL SYSTEMS CO.,LTD.

All rights reserved / MKT-EL-TD-E1605

S = Spray; W = Wipes; L = Liquid; P = Powder; G = Gel

Transducer Disinfectant / Pre-Cleaner / Gel Compatibility Table

Rev. 10

○: FDA Approved X: Not Approved(DO NOT USE) Δ: Not Tested (DO NOT USE)
S = Spray; W = Wipes; L = Liquid; P = Powder; G = Gel

Product Type	Disinfectant									
	Cleanisept wipes	CLEANISEPT® WIPES forte	Protex	Tristel Duo	SteriMax® Sporicide Wipes	SteriMax® Wipes	SteriMax®	UltraSan Broad Spectrum	CaviCide	CaviWipes
Product Type	W	W	S	S	W	W	S	S	S	W
C1-6	O	O	O	O	O	O	O	O	O	O
C1-6T	O	O	O	O	O	X	X	X	X	O
C1-6i	O	O	O	O	O	O	O	O	O	O
SC1-6	O	O	O	O	O	O	O	O	O	O
C5-8	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
SC1-4HS	O	O	O	O	O	O	O	O	O	O
SC1-4H	O	O	O	O	O	O	O	O	O	O
SC1-6H	O	O	O	O	O	O	O	O	O	O
L3-8	O	O	O	O	O	O	O	O	O	O
L3-8i	O	O	O	O	O	O	O	O	O	O
L3-12	O	O	O	O	O	O	O	O	O	O
L3-12T	O	O	O	O	O	X	X	X	X	O
L3-12i	O	O	O	O	O	O	O	O	O	O
L3-12H	O	O	O	O	O	O	O	O	O	O
L3-12HWD	O	O	O	O	O	O	O	O	O	O
L3-12X	O	O	O	O	O	O	O	O	O	O
L8-17	O	O	O	O	O	O	O	O	O	O
L8-17i	O	O	O	O	O	O	O	O	O	O
L8-17X	O	O	O	O	O	O	O	O	O	O
MP1-5X	O	O	O	O	O	O	O	O	O	O
SP1-5	O	O	O	O	O	O	O	O	O	O
SP1-5i	O	O	O	O	O	O	O	O	O	O
SP3-8	O	O	O	O	O	O	O	O	O	O
SP1-5X	O	O	O	O	O	O	O	O	O	O
VC1-6	O	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
SVC1-6	O	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
SVC1-6H	O	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
E3-10	O	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
EN3-10	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
EV3-10	O	O	O	O	O	O	O	O	O	O
EV3-10T	O	O	O	O	O	O	O	O	O	O
EC3-10	O	O	O	O	O	O	O	O	O	O
EC3-10T	O	O	O	O	O	O	O	O	O	O
E3-10H	O	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
EC3-10H	O	O	O	O	O	O	O	O	O	O
EV3-10H	O	O	O	O	O	O	O	O	O	O
IO3-12	O	O	O	O	O	O	O	O	O	O
CW2.0	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
CW5.0	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
SVC1-6	O	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
IO8-17	O	O	O	O	O	X	X	X	X	O
EV3-10X	O	O	O	O	O	X	X	X	X	O
EC3-10X	O	O	O	O	O	X	X	X	X	O
VE3-10H	Δ	Δ	Δ	Δ	Δ	Δ	O	O	O	O
C5-8N	O	O	O	O	O	X	X	X	X	O
C5-8NT	O	O	O	O	O	X	X	X	X	O
SP1-5T	O	O	O	O	O	Δ	O	O	O	O
SP3-8T	O	O	O	O	O	X	X	X	X	O
IO8-17T	O	O	O	O	O	X	X	X	X	O

⚠ CAUTION ⚠

! Do not wipe the cable, strain relief, and connector of the transducer with the disinfectant, as it can damage and/or discolor those parts of the transducer.

! Do not immerse transducers for longer than one hour, unless they are sterilizable. ⚠

Global ALPINION Headquarters

Verdi Tower, 72, Digital-ro 26-gil, Guro-gu, Seoul, 152-848, Korea [Homepage / www.alpinion.com](http://www.alpinion.com)

e-mail / marketing@alpinion.com

TEL +82-2-3282-0900 FAX +82-2-851-5591

Copyright©2016 ALPINION MEDICAL SYSTEMS CO.,LTD.

All rights reserved / MKT-EL-TD-E1605

S = Spray; W = Wipes; L = Liquid; P = Powder; G = Gel

Transducer Disinfectant / Pre-Cleaner / Gel Compatibility Table

Rev. 10

o: FDA Approved X: Not Approved(DO NOT USE) Δ: Not Tested (DO NOT USE)
S = Spray; W = Wipes; L = Liquid; P = Powder; G = Gel

Product Type	Pre-Cleaner											
	T-spray	T-spray II	Enzol	Theracide Plus(SSN)	Transeptic	Septihol	Metrizyme	Klenzyme	McKesson	Cidezyme	Endozyme	EmPower
C1-6	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
C1-6T	Δ	Δ	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	O
C1-6i	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
SC1-6	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
C5-8	O	O	Δ	Δ	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ
SC1-4HS	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
SC1-4H	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
SC1-6H	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
L3-8	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
L3-8i	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
L3-12	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
L3-12T	Δ	Δ	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	O
L3-12i	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
L3-12H	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
L3-12HWD	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
L3-12X	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
L8-17	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
L8-17i	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
L8-17X	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
MP1-5X	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
SP1-5	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
SP1-5i	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
SP3-8	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
SP1-5X	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
VC1-6	Δ	O	O	O	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ
SVC1-6	Δ	O	O	O	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ
SVC1-6H	Δ	O	O	O	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ
E3-10	O	Δ	Δ	Δ	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ
EN3-10	O	Δ	Δ	Δ	Δ	Δ	O	O	O	O	Δ	Δ
EV3-10	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
EV3-10T	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
EC3-10	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
EC3-10T	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
E3-10H	O	Δ	Δ	Δ	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ
EC3-10H	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
EV3-10H	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
IO3-12	Δ	O	O	O	O	O	Δ	Δ	Δ	Δ	O	O
CW2.0	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
CW5.0	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
SVC1-6	Δ	O	O	O	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ
IO8-17	Δ	Δ	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	O
EV3-10X	Δ	Δ	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	O
EC3-10X	Δ	Δ	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	O
VE3-10H	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
C5-8N	Δ	Δ	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	O
C5-8NT	Δ	Δ	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	O
SP1-5T	Δ	Δ	Δ	Δ	O	O	O	O	O	O	Δ	Δ
SP3-8T	Δ	Δ	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	O
IO8-17T	Δ	Δ	O	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	O

※ CAUTION ※

! Do not wipe the cable, strain relief, and connector of the transducer with the disinfectant, as it can damage and/or discolor those parts of the transducer.

! Do not immerse transducers for longer than one hour, unless they are sterilizable. ☒

Global ALPINION Headquarters

Verdi Tower, 72, Digital-ro 26-gil, Guro-gu, Seoul, 152-848, Korea [Homepage / www.alpinion.com](http://www.alpinion.com)

e-mail / marketing@alpinion.com

TEL +82-2-3282-0900 FAX +82-2-851-5591

Copyright©2016 ALPINION MEDICAL SYSTEMS CO.,LTD.

All rights reserved / MKT-EL-TD-E1605

S = Spray; W = Wipes; L = Liquid; P = Powder; G = Gel

Transducer Disinfectant / Pre-Cleaner / Gel Compatibility Table

Rev. 10

○: FDA Approved X: Not Approved(DO NOT USE) Δ: Not Tested (DO NOT USE)
S = Spray; W = Wipes; L = Liquid; P = Powder; G = Gel

Product Type	Gel								Device
	Aquasonic 100	Soflean Gel	Care Sonic Gel	ULTRASOUND GEL	Ultrasound Transmission gel	Scan	Sonogel	Super sonic	Trophon® EPR
Product Type	G	G	G	G	G	G	G	G	
C1-6	○	○	○	○	Δ	Δ	Δ	○	○
C1-6T	○	Δ	Δ	○	○	Δ	Δ	○	○
C1-6i	○	○	○	○	Δ	Δ	Δ	○	○
SC1-6	○	○	○	○	Δ	Δ	Δ	○	○
C5-8	○	Δ	Δ	Δ	Δ	○	○	Δ	Δ
SC1-4HS	○	○	○	○	Δ	Δ	Δ	○	○
SC1-4H	○	○	○	○	Δ	Δ	Δ	○	○
SC1-6H	○	○	○	○	Δ	Δ	Δ	○	○
L3-8	○	○	○	○	Δ	Δ	Δ	○	○
L3-8i	○	○	○	○	Δ	Δ	Δ	○	○
L3-12	○	○	○	○	Δ	Δ	Δ	○	○
L3-12T	○	Δ	Δ	○	○	Δ	Δ	○	○
L3-12i	○	○	○	○	Δ	Δ	Δ	○	○
L3-12H	○	○	○	○	Δ	Δ	Δ	○	○
L3-12HWD	○	○	○	○	Δ	Δ	Δ	○	○
L3-12X	○	○	○	○	Δ	Δ	Δ	○	○
L8-17	○	○	○	○	Δ	Δ	Δ	○	○
L8-17i	○	○	○	○	Δ	Δ	Δ	○	○
L8-17X	○	○	○	○	Δ	Δ	Δ	○	○
MP1-5X	○	○	○	○	Δ	Δ	Δ	○	○
SP1-5	○	○	○	○	Δ	Δ	Δ	○	○
SP1-5i	○	○	○	○	Δ	Δ	Δ	○	○
SP3-8	○	○	○	○	Δ	Δ	Δ	○	Δ
SP1-5X	○	○	○	○	Δ	Δ	Δ	○	○
VC1-6	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	○
SVC1-6	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	○
SVC1-6H	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	○
E3-10	○	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
EN3-10	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
EV3-10	○	○	○	○	Δ	Δ	Δ	○	X
EV3-10T	○	○	○	○	Δ	Δ	Δ	○	X
EC3-10	○	○	○	○	Δ	Δ	Δ	○	○
EC3-10T	○	○	○	○	Δ	Δ	Δ	○	○
E3-10H	○	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
EC3-10H	○	○	○	○	Δ	Δ	Δ	○	○
EV3-10H	○	○	○	○	Δ	Δ	Δ	○	X
IO3-12	○	○	○	○	Δ	Δ	Δ	○	○
CW2.0	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
CW5.0	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ
SVC1-6	Δ	Δ	Δ	Δ	Δ	Δ	Δ	Δ	○
IO8-17	○	Δ	Δ	○	○	Δ	Δ	○	○
EV3-10X	○	Δ	Δ	○	○	Δ	Δ	○	X
EC3-10X	○	Δ	Δ	○	○	Δ	Δ	○	○
VE3-10H	Δ	Δ	○	○	○	Δ	Δ	○	○
C5-8N	○	Δ	Δ	○	○	Δ	Δ	○	○
C5-8NT	○	Δ	Δ	○	○	Δ	Δ	○	○
SP1-5T	Δ	○	○	○	○	Δ	Δ	○	○
SP3-8T	○	Δ	Δ	○	○	Δ	Δ	○	Δ
IO8-17T	○	Δ	Δ	○	○	Δ	Δ	○	○

※ CAUTION ※

! Do not wipe the cable, strain relief, and connector of the transducer with the disinfectant, as it can damage and/or discolor those parts of the transducer.

! Do not immerse transducers for longer than one hour, unless they are sterilizable. ☐

Global ALPINION Headquarters

Verdi Tower, 72, Digital-ro 26-gil, Guro-gu, Seoul,152-848, Korea [Homepage / www.alpinion.com](http://www.alpinion.com)

e-mail / marketing@alpinion.com

TEL +82-2-3282-0900 FAX +82-2-851-5591

Copyright©2016 ALPINION MEDICAL SYSTEMS CO.,LTD.

All rights reserved / MKT-EL-TD-E1605

S = Spray; W = Wipes; L = Liquid; P = Powder; G = Gel